
BISCO

CE 0459

TheraCem® Ca

Self-Adhesive Resin Cement

Instructions for Use

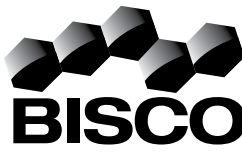BISCO, Inc.
1100 W. Irving Park Rd.
Schaumburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

BISCO France
208, allée de la Coudeulotte
13680 Lançon de Provence
France
33-4-90-42-92-92

Patent Pending

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

en TheraCem® Ca
Self-Adhesive Resin Cement

GENERAL INFORMATION
TheraCem Ca is a self-curing, light-cure, dual-cure resin luting cement that is exclusively formulated for luting crowns, bridges, inlays, onlays and posts (pre-fabricated metal and non-metallic posts, as well as cast posts). TheraCem Ca is not a restorative, calcium-releasing, luting cement which requires no etching, no priming or bonding of the prepared surfaces. It is easy-to-use, requires only a short chair time, and provides a good bond to most dental substrates. TheraCem Ca is available in a natural shade. It is radiopaque, allowing for easy identification on radiographs.

Indications for Use:
Use TheraCem Ca to cement the following:
1. Metal crowns, bridges, inlays, onlays (includes porcelain-fused-to-metal and composite-to-metal)
2. Porcelain, Ceramic Crowns, inlays, and onlays (includes alumina and zirconia)
3. Metal crowns, bridges, inlays, and onlays (resin-based composite/polymethyl methacrylate hybrid)
4. Metal (prefabricated or cast) and non-metallic/ endodontic posts
5. Insulated supported restorations

Contraindications:
TheraCem Ca is not recommended for veneer cementation.

NOTE: It is recommended to use CHOICE™ 2 for veneer cementation.

Warnings:
• Avoid contact with the skin; unpolymerized methacrylate resins may cause skin sensitization in susceptible persons. In case of contact, wash skin with soap and water.
• Saliva and oral fluid contamination will seriously compromise dentin bonding.

Cautions:
Cross-contamination:
• Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
• Use of commonly available hygienic protective covering to avoid any contamination of sprays during treatment is recommended. If spraying becomes contaminated, discard. Do not clean or disinfect.

Precautions:
• Working and Setting Times will be accelerated in the warm intraoral environment.
• Do not use hydrogen peroxide, EDTA, or eugenol containing materials prior to the adhesive procedure as these materials may inhibit bonding.

• Since ceramics and porcelain vary greatly, it is important to consult with the ceramic/porcelain manufacturer or your laboratory for instructions on proper surface treatment and proper prep design for use with TheraCem Ca.
• Unpolymerized resins may cause skin sensitization in susceptible persons. Avoid contact with the skin. In case of contact with the skin, wash thoroughly with soap and water.
• To prevent polymerization or clogging, the Dual-Syringe static mixing tip should be kept in the seal application. If used intraorally, it should be wiped with disinfectant prior to storage.
• See individual component labels for specific expiration dates.
• Safety data sheet available at www.bisco.com.

TECHNICAL INFORMATION
Working Time: Minimum 1 minute (includes mixing) at 22°C (71.6°F)
Setting Time: Maximum 1 minute at 22°C (71.6°F)
Working time and setting time may vary based on storage conditions, temperature, humidity, etc.

INSTRUCTIONS FOR USE
For: Crowns, Bridges, Inlays and Onlays
Preparation Procedures:
NOTE: TheraCem Ca is a self-adhesive cement which does not require etching or priming of the prepared surfaces.
1. Remove the temporary restoration and all temporary cement. **HINT:** To aid in the removal of excess cement, initially light cure the margins for 2-3 seconds.
2. Clean the preparation with enamel prep. Rise thoroughly.
3. Clean the preparation with a strong stream of air on the preparation for 3 to 5 seconds. Do not desiccate. (Protect the preparation from contamination. When applicable, use of a rubber dam is recommended.)

Remove the syringe cap or remove and discard the mixing tip from the dual-syringe.
2. Dispense a small amount of material onto a mixing pad to eliminate any voids in each chamber of the dual-syringe.
3. Attach a mixing tip to the dual-syringe by lining up the key and keyway. Then turn the brown locking mechanism clockwise.
4. Pressing the plunger will mix and dispense the TheraCem Ca. (See "TECHNICAL INFORMATION" Section for Working and Setting Time information.)

Cementation Procedures
1. Apply cement to the internal surfaces of the restoration. For inlays, to ensure a proper fit apply the cement to the both preparation.
2. After the restoration and remove any excess cement. **HINT:** To aid in the removal of excess cement, initially light cure the margins for 2-3 seconds.
3. Light cure for 20-30 seconds, or allow to self cure. (When using self-cured mode, light curing the margins even briefly will expedite the final set.)

HELPFUL HINTS
• TheraCem Ca left on a mixing pad will not appear to set. This is due to oxygen inhibition of the polymerization process. Rest assured the cement will polymerize (set) in a normal fashion under the restoration.

For: Posts (pre-fabricated, metal, cast and non-metallic posts)
Preparation Procedures:
1. Prepare the post space per post manufacturer's instructions. Etching and bonding agents are not required.
2. Clean post and remove any excess cement. **HINT:** To aid in the removal of excess cement, initially light cure the post for 2-3 seconds.
3. Clean post per manufacturer's instructions.

Cement Mixing Procedures
1. Remove the syringe cap or remove and discard the mixing tip from the dual-syringe.
2. Dispense a small amount of material onto a mixing pad to eliminate any voids in each chamber of the dual-syringe.
3. Attach a mixing tip to the dual-syringe by lining up the key and keyway. Then turn the brown locking mechanism clockwise.
4. Pressing the plunger will mix and dispense the TheraCem Ca. (See "TECHNICAL INFORMATION" Section for Working and Setting Time information.)

Cementation Procedures
1. For post cementation, using the root canal mixing tip inject TheraCem Ca into the canal. Start at the apical end of the canal and withdraw the tip slowly while keeping the tip in the cement to minimize voids.
2. Seal the apical end of the post with cement.
3. Seal the post and remove any excess cement. **HINT:** To aid in the removal of excess cement, initially light cure the cement for 2-3 seconds.
4. Proceed with core build-up procedure.

HYGIENE: Use of commonly available hygienic protective covering to avoid any contamination of the TheraCem Ca cement dual-syringes during the treatment is recommended.
DISPOSAL: Refer to regional provisions relating to waste. In their absence, refer to national or community provisions relating to waste.
STORAGE: Store at room temperature (20°C/68°-75°C/17°F).
WARNING: BISCO, Inc. recognizes its responsibility to replace products that prove to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.
TheraCem Ca is a registered trademark of BISCO, Inc.
CHOICE is a trademark of BISCO, Inc.

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Precautions:
• Working and Setting Times will be accelerated in the warm intraoral environment.
• Do not use hydrogen peroxide, EDTA, or eugenol containing materials prior to the adhesive procedure as these materials may inhibit bonding.
• Since ceramics and porcelain vary greatly, it is important to consult with the ceramic/porcelain manufacturer or your laboratory for instructions on proper surface treatment and proper prep design for use with TheraCem Ca.
• Unpolymerized resins may cause skin sensitization in susceptible persons. Avoid contact with the skin. In case of contact with the skin, wash thoroughly with soap and water.
• To prevent polymerization or clogging, the Dual-Syringe static mixing tip should be kept in the seal application. If used intraorally, it should be wiped with disinfectant prior to storage.
• See individual component labels for specific expiration dates.
• Safety data sheet available at www.bisco.com.

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Zementationsverfahren
1. Für post Zementation, using the root canal mixing tip inject TheraCem Ca into the canal. Start at the apical end of the canal and withdraw the tip slowly while keeping the tip in the cement to minimize voids.
2. Seal the apical end of the post with cement.
3. Seal the post and remove any excess cement. **HINT:** To aid in the removal of excess cement, initially light cure the cement for 2-3 seconds.
4. Proceed with core build-up procedure.

HYGIENE: Use of commonly available hygienic protective covering to avoid any contamination of the TheraCem Ca cement dual-syringes during the treatment is recommended.
DISPOSAL: Refer to regional provisions relating to waste. In their absence, refer to national or community provisions relating to waste.
STORAGE: Store at room temperature (20°C/68°-75°C/17°F).
WARNING: BISCO, Inc. recognizes its responsibility to replace products that prove to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.
TheraCem Ca is a registered trademark of BISCO, Inc.
CHOICE is a trademark of BISCO, Inc.

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Zementationsverfahren
1. Für post cementation, using the root canal mixing tip inject TheraCem Ca into the canal. Start at the apical end of the canal and withdraw the tip slowly while keeping the tip in the cement to minimize voids.
2. Seal the apical end of the post with cement.
3. Seal the post and remove any excess cement. **HINT:** To aid in the removal of excess cement, initially light cure the cement for 2-3 seconds.
4. Proceed with core build-up procedure.

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Cement mixing procedures
1. Remove the syringe cap or remove and discard the mixing tip from the dual-syringe.
2. Dispense a small amount of material on a mixing pad to eliminate any voids in each chamber of the dual-syringe.
3. Attach a mixing tip to the dual-syringe by lining up the key and keyway. Then turn the brown locking mechanism clockwise.
4. Pressing the plunger will mix and dispense the TheraCem Ca. (See "TECHNICAL INFORMATION" Section for Working and Setting Time information.)

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