

ALL-BOND 2® ALL-BOND 2® UNIVERSAL DENTAL Adhesive

Instructions for Use

1 of 2

BISCO Inc. 1100 W. Irving Park Road Schaumburg, IL 60193 U.S.A. 1-847-534-6000 1-800-247-3368

BISCO France 208, allée de la Coudelette 13640 Lançon De Provence Rev. 07/22

IN-000513-13-A Rev. 07/22

Caution: US Federal law restricts this device to sale by or on the order of a licensed dentist

CE 0459

en ALL-BOND 2® UNIVERSAL DENTAL Adhesive
GENERAL INFORMATION
1. Composition
2. Application
3. Storage
4. Shelf life
5. Disposal

de ALL-BOND 2® UNIVERSAL DENTAL Adhesive
ALLBOND 2® ALLBOND 2® UNIVERSAL DENTAL Adhesive
1. Description
2. Composition
3. Application
4. Storage
5. Shelf life
6. Disposal

fr ALL-BOND 2® UNIVERSAL DENTAL Adhesive
INFORMATIONS GÉNÉRALES
1. Composition
2. Application
3. Stockage
4. Durée de conservation
5. Élimination

es ALL-BOND 2® UNIVERSAL DENTAL Adhesive
INFORMACION GENERAL
1. Composición
2. Aplicación
3. Almacenamiento
4. Vida útil
5. Eliminación

it ALL-BOND 2® UNIVERSAL DENTAL Adhesive
INFORMAZIONE GENERALE
1. Composizione
2. Applicazione
3. Conservazione
4. Durata di conservazione
5. Smaltimento

sv ALL-BOND 2® UNIVERSAL DENTAL Adhesive
ALLBOND 2® ALLBOND 2® UNIVERSAL DENTAL Adhesive
1. Beskrivning
2. Sammansättning
3. Användning
4. Lagring
5. Shelfliv
6. Avfallshantering

da ALL-BOND 2® UNIVERSAL DENTAL Adhesive
GENERAL INFORMATION
1. Description
2. Composition
3. Application
4. Storage
5. Shelf life
6. Disposal

de ALL-BOND 2® UNIVERSAL DENTAL Adhesive
ALLBOND 2® ALLBOND 2® UNIVERSAL DENTAL Adhesive
1. Beschreibung
2. Zusammensetzung
3. Anwendung
4. Lagerung
5. Haltbarkeit
6. Entsorgung

sv ALL-BOND 2® UNIVERSAL DENTAL Adhesive
ALLBOND 2® ALLBOND 2® UNIVERSAL DENTAL Adhesive
1. Beskrivning
2. Sammansättning
3. Användning
4. Lagring
5. Shelfliv
6. Avfallshantering

da ALL-BOND 2® UNIVERSAL DENTAL Adhesive
GENERAL INFORMATION
1. Description
2. Composition
3. Application
4. Storage
5. Shelf life
6. Disposal

1. Composition
2. Application
3. Storage
4. Shelf life
5. Disposal

1. Description
2. Composition
3. Application
4. Storage
5. Shelf life
6. Disposal

1. Composizione
2. Applicazione
3. Conservazione
4. Durata di conservazione
5. Smaltimento

1. Composição
2. Aplicação
3. Almacenamiento
4. Vida útil
5. Eliminación

1. Beskrivning
2. Sammansättning
3. Användning
4. Lagring
5. Shelfliv
6. Avfallshantering

1. Beschreibung
2. Zusammensetzung
3. Anwendung
4. Lagerung
5. Haltbarkeit
6. Entsorgung

1. Beskrivning
2. Sammansättning
3. Användning
4. Lagring
5. Shelfliv
6. Avfallshantering

1. Description
2. Composition
3. Application
4. Storage
5. Shelf life
6. Disposal

1. Beschreibung
2. Zusammensetzung
3. Anwendung
4. Lagerung
5. Haltbarkeit
6. Entsorgung

1. Beskrivning
2. Sammansättning
3. Användning
4. Lagring
5. Shelfliv
6. Avfallshantering





