BISCO


SELECT HV[®] ETCH[®] High Viscosity

35% Phosphoric Acid Etchant

SELECT HV[®] ETCH (35%) 0459

SELECT HV[®] ETCH (35%) w/BAC

Instructions for Use

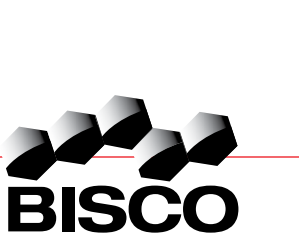IN-190R12

Rev. 9/19

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

SELECT HV[®] ETCH

35% Phosphoric Acid Etchant

GENERAL INFORMATION

BISCO's new **SELECT HV ETCH** allows for precise selective placement. This etchant provides flexibility, yet is just viscous enough to prevent migration. Selective etching provides a deep phosphoric acid coat on the enamel margins while maintaining the benefits of your self-etch adhesive, such as low postoperative sensitivity and moisture tolerance during the cure. The selective or non-selective etching provides a more aesthetic restoration as the enamel margin. The visually brilliant coloring of this new line of etch also makes this phosphoric acid ideal for everyday etching where deep color differentiation is important. BISCO's **SELECT HV ETCH** is an phosphoric acid (H₃PO₄), intended for use on enamel, dentin, and as a cleaning agent for restorative substrates. **SELECT HV ETCH** will effectively remove the smear layer, etch and demineralize enamel and dentin, and remove the necessary components for successful bonding. The etchant returns easily cleanly and quickly, leaving no residue. **SELECT HV ETCH** is also available with benzalkonium chloride (BAC), an antimicrobial agent. Research confirms the development of residual bacterial inhibition zone of up to 10 mm, when challenged (in-vitro) with Actinomycos viscosus **NOTE:** The inclusion of benzalkonium chloride has not been shown to correlate with a reduction in secondary decay in patients. In-vitro clinical studies to evaluate the effects of benzalkonium chloride on oral bacteria or caries have not been performed.

Indications For Use:

- Etching dentin and enamel
- Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended.
- Cleaning agent on dental restorative materials

- Warnings:**
- The phosphoric acid in the etchant is a severe eye and skin irritant. Injury may result if the etchant is allowed to remain on the skin or mucosa for extended periods of time. In case of contact with other tissues, rinse immediately with plenty of water for several minutes.
 - If accidentally splashed into the eye, flush with copious amounts of water and seek medical attention immediately.

Cautions:

Precaution: Product may contain terms that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.

Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:

- Keep away from direct sunlight. Prolonged exposure to sunlight could cause a discoloration of the etchant; this will not compromise the efficacy of the product.
- Test the application of Etchant on a mixing pad or glass slab, BEFORE use on patients so to become familiar with the pressure required to dispense the Etchant from the tip. Never use intracarily before checking flow tip. Never use on patient or on lingual to start flow intracarily, this could result in a subsequent uncontrolled discharge of material, if the etchant tip pack to suction from the back of the syringe. Dispense using new tip and verify flow before use on patient.
- Plophy pastes containing oil or fluoride should not be used as these additives may interfere with etching.
- Use individual component labels for specific expiration dates.
- Always use the etchant with copious amounts of water.
- Safety data sheet available at www.bisco.com.

SELECT HV 30 ML BULK SYRINGE

- Property connect and secure the bulk syringe to the unit dose syringe. System will be fully engaged prior to extruding and containing the effluent purple acid.
- Do not apply force to connect the two systems as it could result in breakage of either syringe connecting Luer.
- Do not apply force to the back of the piston while syringe is capped, has unit dose attached or a without tip on syringe. Pushing back and forth will break about 1/4 inch of the barrel and will extend out the back end of the syringe.
- If bulk syringe is neither capped nor has attached unit dose syringe, material may burst out from tip on syringe.
- Recap bulk syringe while not in use to keep material from drying and getting back into box. Do not forcefully place syringe down with tip in the air as material may splash.
- Do not transfer material back and forth between bulk and unit dose syringe as it may cause air pockets and possible cross-contamination.

NOTE: Be sure to observe the 30 ml bulk syringe

- Material may be contained between the piston and wall of barrel. Product is still in usable condition.
- If observed that material does not extrude into unit dose syringe, secure unit syringe onto bulk syringe system and draw the barrel back to suction the tip of the bulk syringe. Dispense using unit dose syringe, and release air. Repeat as necessary until material begins to extrude into unit dose syringe.

NOTE: Be sure to observe the Unit Dose Syringe

- Unit dose syringe plunger may pop out back if fully loaded. Gently pull back on unit dose syringe plunger to extrude material and dispense.
- Control dispensing tip securely, which may impede result in leakage or accurate spacing.
- Single use only. Otherwise may result in cross-contamination. Discard after use.

If using **SELECT HV ETCH** 30 ml bulk syringe system, proceed as follows:

- Attach the pre-attached unit dose syringe to the back of the bulk syringe by turning the Luer lock to the unit dose syringe lightly on to the male thread of bulk syringe system syringe.

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© 2012 Bisco Dental School, Restald Effect of 1 and 2% Benzocaine Chloride Incorporated into an Ethanol on the Susceptibility of Actinomyces viscosus T147, 1993