

UNI-ETCH




Uni-Etch is a 32% semi-gel phosphoric etch. Uni-Etch is used to etch the tooth structure prior to bonding adhesives, composites, or sealants. It is specifically formulated to be easily applied to larger surface area and rinse cleanly with no residue. Uni-Etch is ideal for the total-etch technique.

Benefits of Uni-Etch



Unparalleled Bond

Creates microretentive surfaces that are necessary for successful bonding. Published research proves that BISCO etchants produce higher bond strengths to (wet or dry) dentin¹ and enamel.

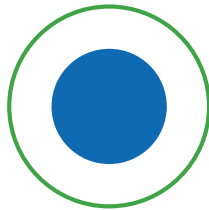

Blue Color

For easy visualization and contrast.



Easy Wash Off

Washes off easily without leaving residue.

Indications for Use



Etching Dentin and Enamel
15 seconds



Total-Etch Technique
15 seconds



Cleaning Agent on Dental Restorative Materials
30 seconds

Did you know?

All-Bond Universal® and Uni-Etch are a Perfect Pair!

All-Bond Universal is a light-cured, universal adhesive compatible with all bonding techniques. This makes it the perfect adhesive to use with Uni-Etch in the total-etch technique!

- ✓ Strong.
- ✓ Reliable.
- ✓ Simple.



Ordering Information

Refills

- Uni-Etch Syringe Pack**..... E-5502EM
4 Syringes Uni-Etch (5g ea.), 60 Disposable Syringe Tips, Instructions
- Uni-Etch Bulk Bottle** E-5637E
1 Bulk Bottle Uni-Etch (30g), Instructions
- Uni-Etch Syringe** E-5632EM
1 Syringe Uni-Etch (5g), 15 Disposable Syringe Tips, Instructions
- Uni-Etch 125 Light Blue Disposable Tips (25 Gauge)** X-80611N
125 ct. Light Blue Phosphoric Etchant Syringe Tips (25 Gauge) used with Uni-Etch

1. Kanca, J.J. "Etchant composition and bond strength to dentin". Am J Dent 1993;6:162-164

For more information,
visit www.bisco.com or
email intl@bisco.com

